


AnyRidge Drilling *GUIDE*

D1

Implant Diameter	Core Dia	WIDEST Thread Dia	SMOOTH Insertion										TIGHT Insertion																
			2.0	2.9	3.3	CD ₄	*3.8	4.3	4.8	5.4	5.9	CD ₆	2.0	2.9	3.3	CD ₄	4.3	CD ₅	4.8	5.4	5.9								
3.5	2.8	4.0	2.0	2.9	3.3	CD ₄	*3.8					2.0	2.9	3.3	CD ₄														
4	3.3	4.4	2.0	2.9	3.3	3.8	CD ₅	*4.3				2.0	2.9	3.3	3.8	CD ₅													
4.5	3.3	4.9	2.0	2.9	3.3	3.8	4.3	CD ₅	*4.8			2.0	2.9	3.3	3.8	4.3	CD ₅												
↓										*If very dense bone																			
6	4.8	6.4	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9	CD ₆		2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9									
6.5	4.8	6.9	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9	CD ₆		2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9									

CORTICAL DRILLS are ALWAYS NEEDED for D1 bone. Oak Wood

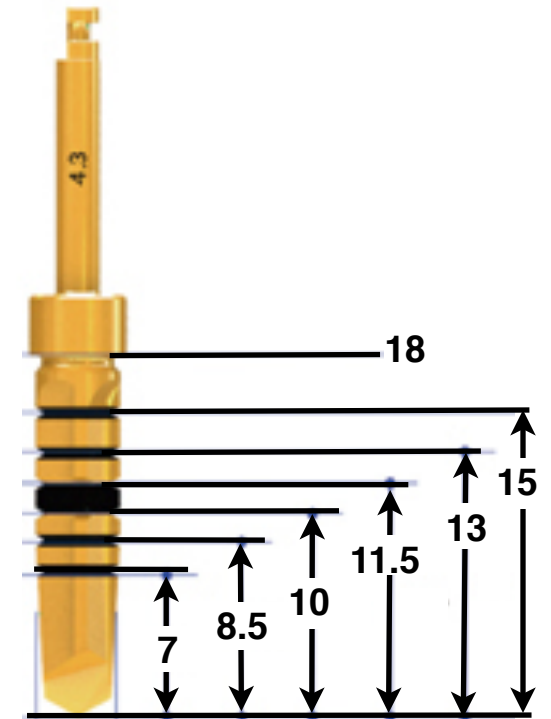

D2

Implant Diameter	Core Dia	WIDEST Thread Dia	SMOOTH Insertion										TIGHT Insertion															
			2.0	2.9	3.3	CD ₄	*3.8	4.3	4.8	5.4	5.9	CD ₆	2.0	2.9	3.3	CD ₄	4.3	CD ₅	4.8	5.4	5.9							
3.5	2.8	4.0	2.0	2.9	3.3	CD ₄	*3.8					2.0	2.9	3.3														
4	3.3	4.4	2.0	2.9	3.3	3.8	CD ₅	*4.3				2.0	2.9	3.3	3.8													
4.5	3.3	4.9	2.0	2.9	3.3	3.8	4.3	CD ₅	*4.8			2.0	2.9	3.3	3.8	4.3												
5	3.3	5.4	2.0	2.9	3.3	3.8	4.3	4.8	CD ₅	*5.4		2.0	2.9	3.3	3.8	4.3	4.8											
↓	5.5 & 7-8 Diameter Not Recommend for D2										*If very dense bone																	
6	4.8	6.4	2.0	2.9	3.3	3.8	4.3	4.8	5.4	CD ₆	*5.9		2.0	2.9	3.3	3.8	4.3	4.8	5.4									
6.5	4.8	6.9	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9	CD ₆		2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9								

CORTICAL DRILLS are OFTEN NEEDED for D2 bone. Spruce Wood

D3


Implant Diameter	Core Dia	Widest Thd Dia	SMOOTH Insertion								TIGHT Insertion																
3.5	2.8	4.0	2.0	2.9	CD 4 Opt.									2.0	2.9												
4	3.3	4.4	2.0	2.9	3.3	CD 5 Opt.								2.0	2.9	3.3											
4.5	3.3	4.9	2.0	2.9	3.3	3.8	CD 5							2.0	2.9	3.3	3.8										
5	3.3	5.4	2.0	2.9	3.3	3.8	4.3	4.8 Opt.						2.0	2.9	3.3	3.8	4.3									
5.5	3.3	5.9	2.0	2.9	3.3	3.8	4.3	4.8	5.4 Opt.					2.0	2.9	3.3	3.8	4.3	4.8								
6	4.8	6.4	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9 Opt.				2.0	2.9	3.3	3.8	4.3	4.8								
6.5	4.8	6.9	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9 Opt.				2.0	2.9	3.3	3.8	4.3	4.8								
7	4.8	7.4	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9				2.0	2.9	3.3	3.8	4.3	4.8	5.4							
7.5	4.8	7.9	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9				2.0	2.9	3.3	3.8	4.3	4.8	5.4							
8	4.8	8.4	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9				2.0	2.9	3.3	3.8	4.3	4.8	5.4							

CORTICAL DRILLS are OFTEN OPTIONAL for D3 bone. Balsa Wood

NOTE: Opt.= Optional

D4

Implant Diameter	Core Dia	Widest Thd Dia	SMOOTH Insertion								TIGHT Insertion																
3.5	2.8	4.0	2.0	2.9										2.0	2.9												
4	3.3	4.4	2.0	2.9	3.3 Opt.									2.0	2.9												
4.5	3.3	4.9	2.0	2.9	3.3 Opt.									2.0	2.9												
5	3.3	5.4	2.0	2.9	3.3 Opt.									2.0	2.9												
5.5	3.3	5.9	2.0	2.9	3.3	3.8 Opt.								2.0	2.9	3.3											
6	4.8	6.4	2.0	2.9	3.3	3.8	4.3	4.8	5.4 Opt.					2.0	2.9	3.3	3.8	4.3	4.8								
6.5	4.8	6.9	2.0	2.9	3.3	3.8	4.3	4.8	5.4 Opt.					2.0	2.9	3.3	3.8	4.3	4.8								
7	4.8	7.4	2.0	2.9	3.3	3.8	4.3	4.8	5.4 Opt.					2.0	2.9	3.3	3.8	4.3	4.8								
7.5	4.8	7.9	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9 Opt.				2.0	2.9	3.3	3.8	4.3	4.8	5.4							
8	4.8	8.4	2.0	2.9	3.3	3.8	4.3	4.8	5.4	5.9 Opt.				2.0	2.9	3.3	3.8	4.3	4.8	5.4							

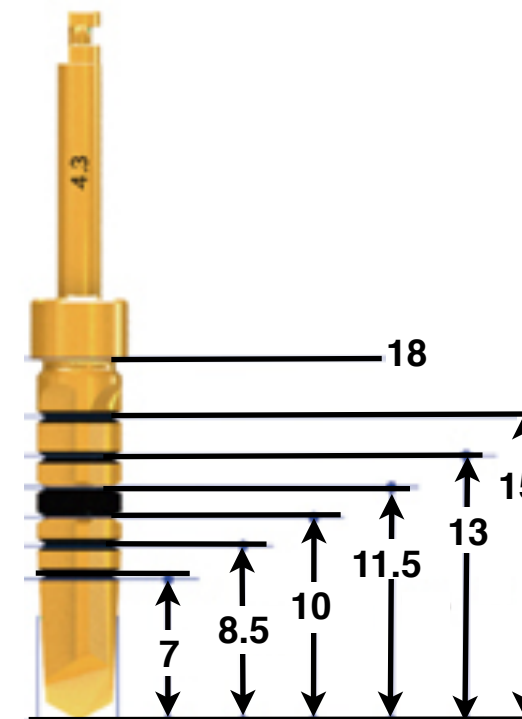

CORTICAL DRILLS are NOT TYPICALLY NEEDED for D4 bone. Coarse Styrofoam